

ELECTROCHEM TUBING FOR MEDICAL USE

INSTRUCTIONS FOR USE

1. Following instructions apply to the products in form of tubing for medical use of "Dr. DELIBALTAS Ltd. - electrochem".
2. The products we manufacture for medical use in the category of Medical Disposables are SINGLE USE under European legislation (Directive 93/42/EEC for Medical Devices).
3. The products of Class I, in accordance with Directive 93/42/EEC, are delivered non-sterile. For the sterilization method please contact us so that we can send you the official brochure of our company, containing Description, Specifications, Properties and Methods of Sterilization in English, or visit our web page www.electrochem.eu and download the brochure "Medical Tubing Portfolio" in digital form. On our website you can find brochures in electronic (digital) format available in other languages.
4. Our products are designed to fit for purposes, which are fully described in the official brochures of our Company.
5. Our products are not designed to be used as an implant for therapeutic, restorative, prosthetic, improve aesthetics, etc. or for purposes not outlined in official brochures of our Company.
6. Our products are not recommended for experiments of any kind, beyond the use of methods which are described in official brochures of our Company.
7. All our products leave our facilities qualitatively tested and packed air-tightly. If for any reason the packaging is open or torn do not use the product. Contact your supplier immediately.
8. Open each packaging with care.
9. Especially in the case of silicone products, please take into account that due to the positive electrostatic charge, which is typical property of silicone elastomers, traces of dust and small particles (e.g. fibers etc) may be town. It is recommended that the installation of the product will be done on a clean surface and in controlled work environment, and the completion of work by the operator will be without delay.
10. Do not injure the surface of the product in any way (e.g., sharp objects, etc.) and not change its shape (e.g. knots, twists, crushing the material in any way etc).
11. **Products in the form of STRAIGHT tube in rolls:** if you wish to use a smaller length than that of the roll, cut the tube to the desired length, use a new pair of scissors and make the cut vertically to the longitudinal axis and not sideways or along the tube.
12. **Products in the form of BUBBLE tubing in rolls:** if you wish to use a smaller length than that of the roll, cut to the desired length at the appropriate point after selecting the appropriate diameter. Preferably select the area of enlargement of the diameter to achieve the best tight fit. For this reason, mark with a pen a point, and use a new pair of scissors and make the cut vertically in the longitudinal tube axis and not sideways or along the tube.
13. **Products in the form of sterilized CONNECTION TUBING or SET ready for use:** use without additional interventions, assuring that the poche is unharmed and in best condition.
14. **Products contained in DISPENSER BOX:** open with care the circular part on the top of the carton box. Read first the instructions inside and on the top. Cut the sealed bag in the center with caution using a scissor. Find the roll's end near center and pull it out. Note: As you continue to pull the rest of the tubing out, it will unreel very easily till the end of the entire roll. Follow the instructions as per Paragraph No. 11 and/or 12.
15. **Products in rolls, used for CUTTING Machines by unrolling the roll:** if you need to unroll this kind of product until it's total end, please handle with care. Do not shake the bags. Important : be sure that the logo is on the top side. Then cut with precaution the upper part of the PE bag using a scissor, until it produces a clear square, which dimensions are equal to the bag's dimensions. Take away the cutted part. Hold the start of the roll, which is marked with a sticker or band. Ensure that the machine is fed smoothly.
16. Used products with no exceptions should be immediate and controlled discharged in accordance with regulations set by the local legislation for rejecting Used Medical Supply Hardware.
17. If for any reason you believe that the packaging of the product is not authentic (e.g. absence of the printed logo of our Company from the packaging or sticker that does not meet the design as you know), please contact us as soon as possible to the phones +49 6966554388 and +30 2310 796680 or email us at: infode@electrochem.eu and info@electrochem.eu